


Cone Beam CT - A Primer for Orthopedic Surgeons

CurveBeam AI Cone Beam CT (CBCT) systems are designed to capture three-dimensional weight bearing and non-weight bearing volumetric images of the body extremities.

Cone Beam CT images are initially acquired as two-dimensional projections, using a rotating gantry with a fixed-anode X-Ray tube firing a pulsed X-Ray beam on to a flat panel detector. The gantry rotates 360 degrees and acquires image projections, which are then reconstructed to create a series of axial slices.

The osseous (high contrast) details in CBCT datasets should be at par with (or very close to) conventional medical CT. This is the primary diagnostic objective for CBCT scans.

The default slice thickness for CurveBeam AI datasets ranges from 0.2 - 0.37 mm, depending on the system and field of view setting.

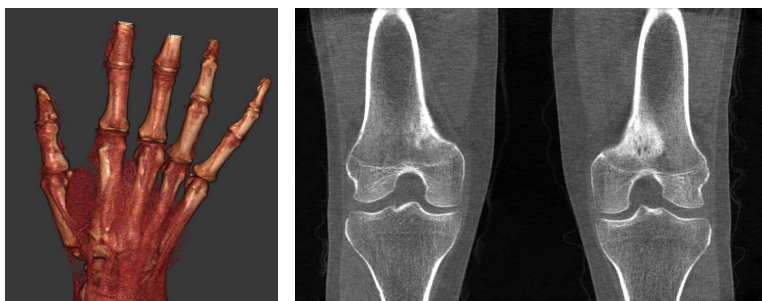

Weight Bearing Advantage

Weight bearing CT (WBCT) imaging systems scan the patient while he or she is standing naturally. The resulting study gives orthopedic surgeons a 3D view of bone morphology, alignment, and joint spaces in the lower extremities.

Lower extremity specialists may elect to order WBCT scans for the same indications they would have otherwise requested weight bearing radiographs. The indications for weight bearing CT are broader than the indications for conventional medical CT.


Indications for weight bearing CT include:

- Arthritic joint space evaluation
- Hindfoot alignment evaluation
- Patellar alignment and instability
- Syndesmosis
- Forefoot evaluation for hallux valgus (bunion)
- Flat foot evaluation
- Post-surgical fusion evaluation

Improved FX Detection Rate

CBCT provides 35% improved simple fracture detection rates and 2-fold improved identification of complex fractures over X-Ray¹, which helps in the evaluation of the fracture healing process which X-Ray can over or under-estimate². In addition, CBCT imaging shows a higher sensitivity in detection of small bone and joint trauma over X-Ray and may visualize occult fractures³.

HiRise™

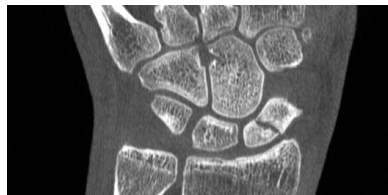

(1) Lodlow, J. Hand-wrist, Knee, and Foot-ankle Dosimetry and image quality measurements of a Novel Extremity Imaging Unit Providing CBCT and 2D Imaging Options. Draft version 1/18/2018.

(2) RSNA Radiologyinfo.org/en/info.cfm?pg=safety-xray

(3) Biswas Debdut et al, Radiation Exposure from Musculoskeletal computerized Tomographic Scans. Journal of Bone & Joint Surgery, Vol 91-A, NO. 8, August, 2009

CPT Codes

- 73200 - CT Upper Extremity w/o contrast
- 73700 - CT Lower Extremity w/o contrast
- 73201 - CT Upper Extremity w/ contrast
- 73701 - CT Lower Extremity w/ contrast
- 76376 - 3D Render with Interpretation Post-Processing

Low Dose

CurveBeam AI systems are specifically designed to expose the patient to radiation dose that is As Low as Reasonably Achievable (ALARA) while acquiring the images. This chart depicts the effective dose to the patient in milliSieverts, depending on the protocol selected. Please note that the mA ranges from 5 - 6.5, and kVp ranges from 100 - 130.

pedCAT

large field of view bilateral foot scan: .006 mSv

InReach

hand/wrist scan: .001 mSv

LineUP

bilateral knee scan: .001 mSv

HiRise

elbow scan: .002 mSv